

DigiModel™

Download and Registration Instructions

How to Download and Register Your DigiModel™ Software in 7 easy steps!

Step 1: Click the *Download DigiModel Free* button

Step 2: In the menu list, click *DigiModel*, then click the *Free Download* button and *Run*.

Step 3: The Install Wizard will guide you through the install process and a DigiModel icon will appear on your desktop



Step 4: Open DigiModel; a dialog box will appear with three choices - select Register/Evaluate

Step 5: A Registration Dialog box will appear with a Product Install code already populated in the first entry space. In the second entry space, enter the email address you use or will use for your OrthoProof USA account. Then click *Contact OrthoProof for Registration*.

Step 6: A new page will open in your browser with a Registration Code; copy and paste that code into the third entry in the Registration Dialog box and click *Ok*.

Step 7: Contact OrthoProof USA to configure the settings in your newly downloaded software **FREE** of charge!